

READY TO USE **Wexicide**

A one-step cleaner disinfectant

Germicidal Detergent
Cleans • Disinfects • Deodorizes

- A powerful one-step cleaner, disinfectant and deodorizer
- Use for disinfection of all hard, non-porous surfaces in Healthcare Facilities including Operating Rooms, Intensive Care, Nurseries, Emergency Areas and Dental Operatories
- Recommended for use by Educational, Athletic and other Institutional Facilities as well as Police and EMS to decontaminate hard inanimate surfaces that have the potential to be contaminated by blood or other potentially infectious material (OPIM)
- Easy to use non-aerosol dispensing cap on bottle
- Totally broad spectrum: TUBERCULOCIDAL, GERMICIDAL, PSEUDOMONACIDAL, FUNGICIDAL AND VIRUCIDAL (see additional technical data on back page)
- Effective in blood serum: Products that are effective in the presence of organic soil such as 5% blood serum are classified as one step cleaner/disinfectants (EPA DIS TSS-2)
- Effective against antibiotic resistant organisms, including MRSA, MRSE and VRE
- Excellent cleaner/detergent

DO NOT MIX WITH OTHER CLEANERS

Meets the current OSHA requirements for cleaning and decontaminating spills of blood and other potentially infectious material (OPIM).

Wexford Labs
The **Disinfectant Solution**

325 Leffingwell Avenue, Kirkwood, Missouri 63122
www.wexfordlabs.com



READY TO USE
CLEANS •
GERMICIDAL, PSEUDOMONACIDAL,
FUNGICIDAL AND VIRUCIDAL
ACTIVE INGREDIENT: ortho-
chloro-*p*-tert-butyl-phenol
OTHER INGREDIENTS: ...
EPA REG. NO. ...
KEEP OUT OF ...
GERMICIDAL ACTIVITY: *S. aureus*, *S. choleraesuis*,
Ps. aeruginosa, *S. typhi*, *S. dysenteriae*,
Enterococcus, Fecal (VRE), TUBERCULOCIDAL
presence of 5% serum at 20°C for 10 minutes
in the presence of 5% serum. (VRE)
simplex Types 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 11, 12, 13, 14, 15, 16, 17, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 28, 29, 30, 31, 32, 33, 34, 35, 36, 37, 38, 39, 40, 41, 42, 43, 44, 45, 46, 47, 48, 49, 50, 51, 52, 53, 54, 55, 56, 57, 58, 59, 60, 61, 62, 63, 64, 65, 66, 67, 68, 69, 70, 71, 72, 73, 74, 75, 76, 77, 78, 79, 80, 81, 82, 83, 84, 85, 86, 87, 88, 89, 90, 91, 92, 93, 94, 95, 96, 97, 98, 99, 100
Use for all disinfectant for hard, non-porous
surfaces, emergency procedures, nurseries,
PRECAUTIONARY STATEMENTS
CAUTION: Causes moderate eye irritation.
Wash thoroughly with soap and water after handling.
FIRST AID - IF IN EYES: Hold eyelids open, flush with copious amounts of water for at least 15 minutes. Remove contact lenses, if present, and continue flushing. Store at room temperature. Do not use in empty container. Use thoroughly. Wexicide
Wexford Labs
The Disinfectant Solution

READY TO USE Wexside

A one-step cleaner disinfectant

Germicidal Detergent

Cleans · Disinfects · Deodorizes

EPA Registration Number 34810-21

- Effective in 5% blood serum
- Contact Time: 10 minutes

Germicidal Qualifications:

Salmonella enterica for effectiveness against Gram-negative bacteria
Staphylococcus aureus for effectiveness against Gram-positive bacteria
Pseudomonas aeruginosa for nosocomial Gram-positive bacterial pathogen

Additional Bacteria:

Methicillin resistant *Staphylococcus aureus* (MRSA),
Methicillin resistant *Staphylococcus epidermidis* (MRSE)
Tetracycline, Novobiocin, Streptomycin resistant *Staphylococcus aureus* (TNSR),
Gentamicin, Methicillin resistant *Staphylococcus aureus* (GRM),
Penicillin resistant *Staphylococcus* (PR),
Enterococcus Faecalis (VRE).

TUBERCULOCIDAL ACTIVITY TEST:

Mycobacterium Bovis

FUNGICIDAL TEST:

Trichophyton Mentagrophytes

VIRUCIDAL ACTIVITY TESTS:

Influenza A3 (Hong Kong)
Pandemic 2009 H1N1 influenza A virus
Adenovirus Type 2
Herpes Simplex, Types 1 and 2
Vaccinia Virus
Human Immunodeficiency Virus, Type 1 (AIDS) (**Contact Time: 1 minute. This contact time will not control other common types of viruses and bacteria.**)
Human Rotavirus
Respiratory Syncytial Virus
Rhinovirus Type 37

FUNGICIDAL TEST:

Trichophyton mentagrophytes



www.wexfordlabs.com
(800) 506-1146